

Emergency Life Support

CPR Prompt® Directions

Adult/Child Manikin



(1.) Remove black neck lock



(2.) Remove adjustable chest piston ("clicker").



(3.) Insert the insertion tool through the neck and out the mouth; attach Adult/Child lung bag to the insertion tool; pull through mouth and neck.



(4.) Pull lung bag through the top of the neck brace; slide head into neck brace; snap the black neck lock back into place; insert the chest piston.




(5.) Adult/Child Manikin is now ready for CPR training and practice.

Infant Manikin



(1.) Remove outer chest plate.

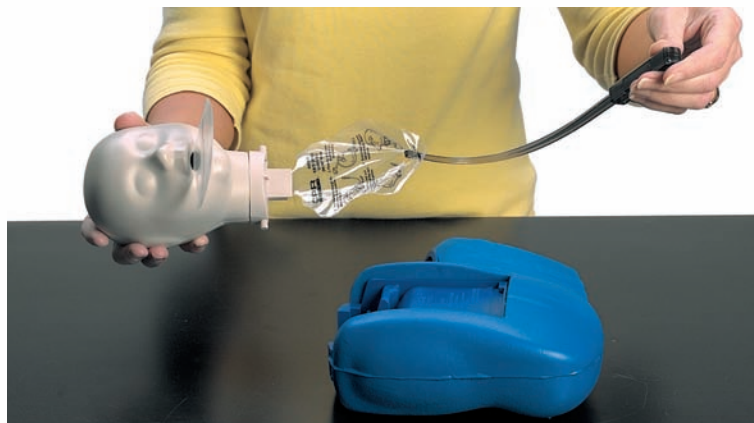

(2.) Same as step #3 of Adult/Child Manikin directions.



(3.) Slide head into neck brace and snap outer chest plate back into place.



(4.) Infant Manikin is now ready for CPR training and practice.